



Crowns

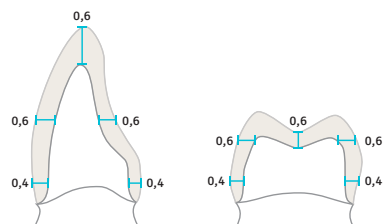

Veneers

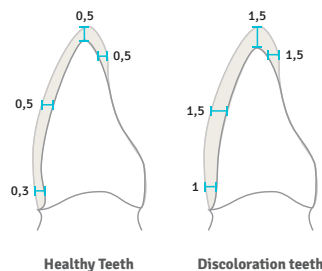

Submerged Implant




Place an abutment with the correct gingival height to raise up the connection as possible.

Advantages:


- Ensure the thickness of the emergency profile in the restoration
- Avoid descementation.

Vertical dimension

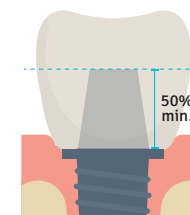


The height of the tibase has to cover the 50% of the height of the G-CAM restoration as a minimum, to ensure the stability of the structure and avoid descementation.


Thickness around the abutment

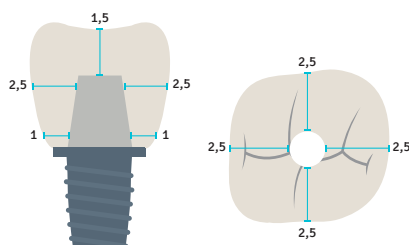


Connector cross section

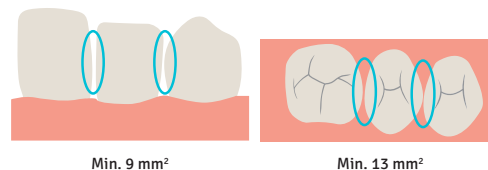

Up to 3 pontics

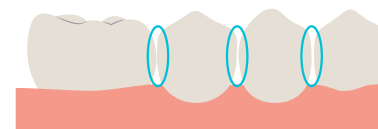


Cantilever

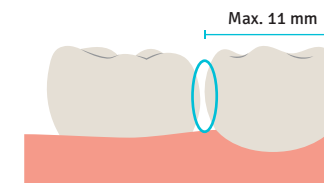


Min. 13 mm² and do not individualize the structure at the lingual or palatal area.

Onlays / Inlays

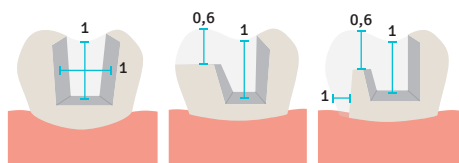

Parallel and defined margins

Milling strategy

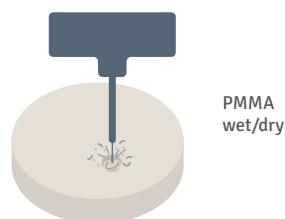

Finishing



G-CAM

Graphene reinforced biopolymer disc for
CAD/CAM drilling

CE
1984

FDA
cleared

GRAPHENE
INTO

Made in
Spain